



ALLEVYN[®] Dressings: Awkward areas application guide

ALLEVYN Dressings* are available in a variety of shapes and sizes which, depending on the wound, can be used on a number of different areas of the body.^{1-16†}

ALLEVYN Dressings for heel



Using an ALLEVYN LIFE or GENTLE BORDER Heel Dressing, remove the central protector film and position the heel onto the dressing, with the upper (malleolus) lobes at right angles to the leg.



Remove the protector film from one upper lobe, fold it onto malleolus and secure in place. Repeat with the other upper lobe.



Remove film from bottom lobe and secure. The dressing should now be in place. Ensure dressing is flush to the skin, smooth edges all around.

ALLEVYN Dressings for hands



Take an ALLEVYN HEEL Dressing and make 4 equidistant incisions into the arch. Incisions should be approximately 1.5cm deep, dependant of finger size.



Place the dressing over the hand. Feed the fingers through the 4 incisions made. Ensure dressing is snug.



To secure apply OPSITE[®] FLEXIFIX Transparent Film Roll to the bottom of the dressing at the base of the palm. This will catch the back and front of the dressing. Ensure OPSITE FLEXIFIX forms a border to the dressing and does not cover ALLEVYN HEEL Dressing completely.

ALLEVYN Dressings for fingers



Use an ALLEVYN ADHESIVE or GENTLE BORDER Dressing (e.g. 7.5cm x 7.5cm). Ensure surrounding skin is dry. Turn the dressing diagonally and place the finger on the corner edge of the dressing.



Lift the dressing over the tip of the finger. Turn the finger over and secure the corners across the finger in turn.



Ensure dressing is flush to the skin without wrinkles. Smooth edges all around.

ALLEVYN Dressings for toes



Take an ALLEVYN square dressing (e.g. 7.5cm x 7.5cm GENTLE BORDER or ADHESIVE Dressing) and turn diagonally. Ensure surrounding skin is dry. Place the toe on the corner edge of the dressing.



Lift the dressing over the tip of the toe. Turn the edges of the dressing across the back of the toe.



Ensure the dressing is flush to the toe. Smooth edges all around.

ALLEVYN Dressings for between toes



Take an ALLEVYN NON-ADHESIVE Dressing 5cm x 5cm. Fold the dressing in half. Make 2 incisions into the dressing 1cm/2cm apart.



Feed the toes into the incisions made. Ensure there is enough dressing between the toes.



Secure in place with OPSITE FLEXIFIX Roll. Ensure OPSITE FLEXIFIX Roll forms a border to the dressing and does not cover ALLEVYN NON-ADHESIVE Dressing completely.

ALLEVYN Dressings for elbows

Bony prominences, such as elbows, are prone to developing wounds.

ALLEVYN ADHESIVE Dressings conform to body contours and are comfortable for the patient.⁸⁻¹⁰

ALLEVYN GENTLE BORDER Dressings allow flexibility and are conformable to awkward areas and areas of the body prone to movement.³⁻⁶



ALLEVYN ADHESIVE SACRUM Dressing



ALLEVYN GENTLE BORDER Multisite Dressing



ALLEVYN GENTLE BORDER Heel Dressing

Removal of silicone adhesive dressings

To remove the dressing, lift one corner of the dressing and slowly peel back until completely removed from the wound.

ALLEVYN Dressings for breast

Breast surgery often creates wounds that need covering with a dressing.

ALLEVYN GENTLE BORDER Multisite Dressings remain soft to the skin and are conformable to fit a wound in any area around the breast.^{3,5,17,18}

ALLEVYN LIFE Dressings are conformable^{1,19} and can be applied to awkward areas of the body.^{1,2}



ALLEVYN GENTLE BORDER Multisite Dressing



ALLEVYN LIFE Dressing

† Please note

Making an incision in an ALLEVYN Dressing may increase the potential for fluid and bacterial breakthrough, and can compromise the bacterial barrier properties provided by the dressing.

ALLEVYN LIFE Dressings

S+N Code	Size	Carton
66801067	10.3cm x 10.3cm	10
66801068	12.9cm x 12.9cm	10
66801069	15.4cm x 5.4cm	10
66801070	21cm x 21cm	10
66801304	25cm x 25.2cm Heel	5
66801306	17.2cm x 17.5cm Sacrum	10
66801307	21.6cm x 23cm Sacrum	10

ALLEVYN GENTLE Dressings

S+N Code	Size	Carton
66802128	5cm x 5cm	10
66802129	10cm x 10cm	10
66802130	10cm x 20cm	10
66802131	15cm x 15cm	10
66802132	20cm x 20cm	10
66802133	20cm x 50cm	2

ALLEVYN GENTLE BORDER Dressings

S+N Code	Size	Carton
66800269	7.5cm x 7.5cm	10
66800270	10cm x 10cm	10
66800900	10cm x 20cm	10
66800264	10cm x 25cm	10
66800265	10cm x 30cm	10
66800272	12.5cm x 12.5cm	10
66800975	15cm x 15cm	10
66800273	17.5cm x 17.5cm	10
66800897	16.8cm x 17.1cm Sacrum	6
66800898	16.8cm x 17.1cm Sacrum	10
66800959	17.1cm x 17.9cm Multisite	10
66801031	21.6cm x 23cm Sacrum	10
66800506	23cm x 23.2cm Heel	5

ALLEVYN GENTLE BORDER LITE Dressings

S+N Code	Size	Carton
66800833	5cm x 5cm	10
66800834	7.5cm x 7.5cm	10
66800835	10cm x 10cm	10
66800836	5.5cm x 12cm	10
66800837	8cm x 15cm	10
66800840	15cm x 15cm	10
66801012	10cm x 20cm	10
66801013	8.6cm x 7.7cm Oval	10
66801014	15.2cm x 13.1cm Oval	10
66801041	8cm x 8.4cm Multisite	10

ALLEVYN ADHESIVE Dressings

S+N Code	Size	Carton
66000043	7.5cm x 7.5cm	10
66000599	10cm x 10cm	10
66000044	12.5cm x 12.5cm	10
66000045	17.5cm x 17.5cm	10
66000744	12.5cm x 22.5cm	10
66000046	22.5cm x 22.5cm	10
66000451	22cm x 22cm Sacrum	10
66000700	17cm x 17cm Sacrum	10

ALLEVYN NON-ADHESIVE Dressings

S+N Code	Size	Carton
66007643	5cm x 5cm	10
66007637	10cm x 10cm	10
66007335	10cm x 20cm	10
66000093	15cm x 15cm	10
66007638	20cm x 20cm	10
66000663	40cm x 70cm	2
66007630	Heel	5
66000773	Heel	10

*ALLEVYN LIFE, ALLEVYN GENTLE BORDER, ALLEVYN GENTLE BORDER LITE, ALLEVYN ADHESIVE, and ALLEVYN NON-ADHESIVE Dressings.

References: 1. Simon D, Bielby A. A structured collaborative approach to appraise the clinical performance of a new product. *Wounds UK*. 2014;10(3):80-87. 2. Stephen-Haynes J, Bielby A, Searle R. The clinical performance of a silicone foam in an NHS community trust. *Journal of Community Nursing*. 2013;27(5):50-59. 3. Hurd T, Gregory L, Jones A, Brown S.A multi-centre in-market evaluation of ALLEVYN Gentle Border. *Wounds UK*. 2009;5(3):32-44. 4. Smith+Nephew 2016. ALLEVYN Gentle Border Heel Gen 2 - physical evaluation. Internal Report. DS/16/465/R. 5. Smith+Nephew 2017. ALLEVYN Gentle Border Gen 2 - Physical Evaluation. Internal Report. DS/16/424/R V3. 6. Rafter L, Reynolds T, Rafter M. An audit of patient outcomes in the management of skin tears using silicone dressings. *Wounds UK*. 2016;12(2):70-78. 7. Stephen-Haynes J, Greenwood M. Clinical evaluation of Allevyn Gentle Border Lite within one health trust. *British Journal of Nursing*. 2011;20(20):S36-S42. 8. Leonard S, McCluskey P, Long S, Butters V, Winter R, Smith G. An evaluation of Allevyn™ Adhesive and Non-Adhesive foam dressings. *Wounds UK*. 2009;5(1):17-26. 9. Bale S, Squires D, Varnon T, Walker A, Benbow M, Harding K. A comparison of two dressings in pressure sore management. *J Wound Care*. 1997;6(10):463-466. 10. Amione P, Ricci E, Topo F, et al. Comparison of Allevyn Adhesive and Biatain Adhesive in the management of pressure ulcers. *J Wound Care*. 2005;14(8):365-370. 11. Smith+Nephew 2015. Cutting of ALLEVYN Variants. Internal Report. DS/14/318/R. 12. Williams C, Young T. Allevyn adhesive. *Br J Nurs*. 1996;5(11):691-693. 13. Foster AVM, Greenhill MT, Edmonds ME. Comparing two dressings in the treatment of diabetic foot ulcers. *Journal of Wound Care*. 1994;3(5):224-228. 14. Smith+Nephew 2015. Cutting of ALLEVYN Variants Internal Report DS/14/318/R. 15. Foster AVM, Greenhill MT, Edmonds ME. Comparing two dressings in the treatment of diabetic foot ulcers. *Journal of Wound Care*. 1994;3(5):224-228. 16. Kurring PA, Roberts CD, Quinlan D. Evaluation of a hydrocellular dressing in the management of exuding wounds in the community. *British Journal of Nursing*. 1994;3(20):1049-1050, 1052-1043. 17. Callaghan R, Chadwick P, Mahoney K, Stephen-Haynes J. Case series evaluation: The use of Allevyn gentle border multisite on chronic wounds. *Wounds UK*. 2012;8(4):110-118. 18. Smith+Nephew 2018. Volunteer Evaluation of Wound Care Dressing Product Features. Internal Report. AWM/AWC/001/v1. 19. Rossington A, Drysdale K, Winter R. Clinical performance and positive impact on patient wellbeing of ALLEVYN Life. *Wounds UK*. 2013;9(4):91-95.

For detailed product information, including indications for use, contraindications, precautions and warnings, please consult the product's applicable Instructions for Use (IFU) prior to use.

